

Hotline Fluid Warmer

- Check water level; it must be between the min and max lines. Use sterile water to fill, if necessary.
- Make sure disposable tubing is plugged into unit
- Turn on power, circulating water will automatically prime the heating ports of the tubing. Nothing should come out of the tubing.
- Now plug you IV or blood set into the proximal end of the tubing.
- **FLUSH THE HOTLINE TUBING WITH YOUR IV SET.** Make sure fluid has emerged from the distal end
- Attach the distal end of the hotline tubing to patient's IV site
- You can use a pressure bag on your IV set or blood bag if you need rapid infusion
- Call biomed if you need more tubing

